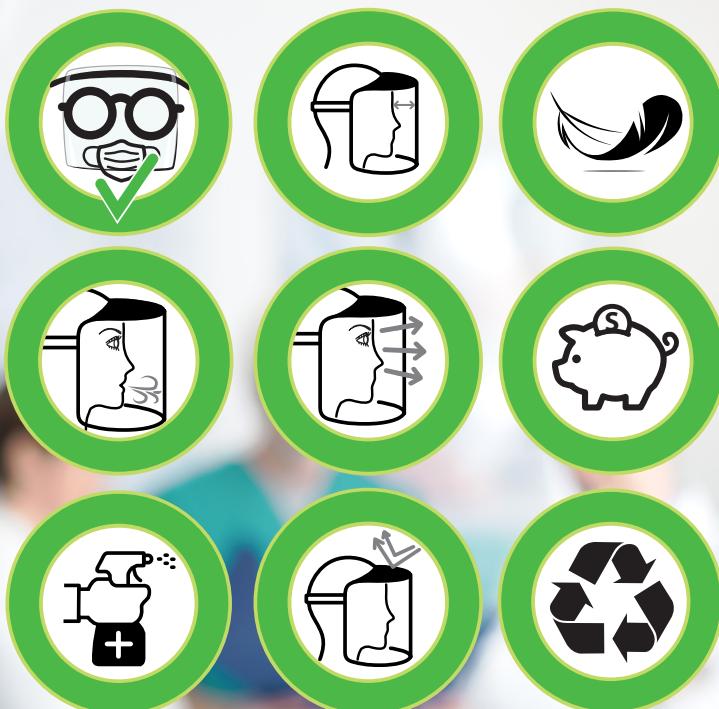
PROTECTIVE VISOR "SHIELD"

DPG01CE
Protective Visor Shield
PP/Poliestere

DPI/PPE/EPI/PSA CAT II - Reg. (UE) 2016/425 - EN 166: 2001

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🇬🇧 Nr. 1 UNBREAKABLE FRONT SUPPORT
Nr. 2 REPLACEABLE SCREENS IN OPTICAL CLASS 1 (cod. DPGR1CE-21)

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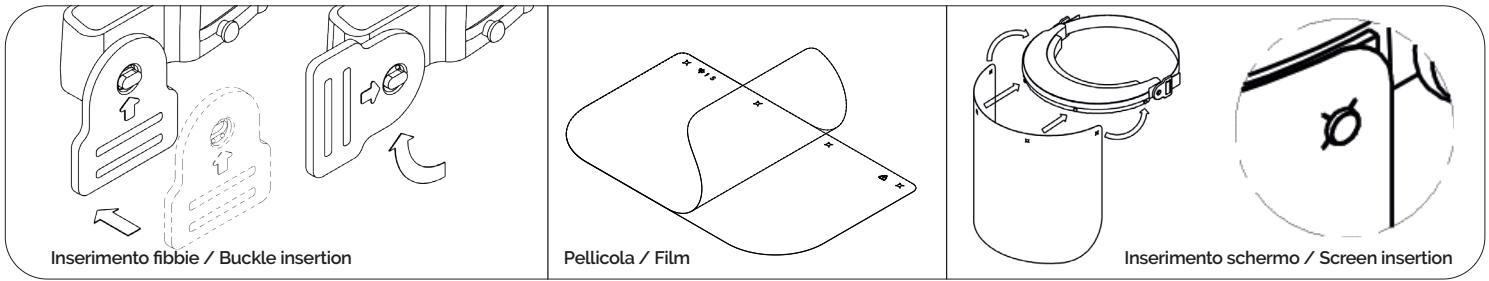
Markings:	Frame EN166 3 S CE	Screen 1 S
	= Manufacturer identification EN166 = European Standard 3 = Protection against liquids (droplets or splashes) S = Increased robustness CE = European conformity marking	= Manufacturer identification 1 = Optical class 1 S = Increased robustness

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USE AND MAINTENANCE INSTRUCTIONS

Before proceeding with the use of the PPE, the user must verify that it is suitable for the intended use. If the PPE is used with other PPE. The user shall assess whether the combination is suitable for use. Wear the PPE before exposing yourself to any risk conditions for which protection is required. In the event of damage during use, the PPE shall be immediately replaced. Do not expose PPE to heat or UV radiation. Materials which may come into contact with the wearer's skin may cause allergic reactions to sensitive individuals. Scratched or damaged eyepieces should be replaced. This PPE is not designed to protect against high-speed particles at both ambient and extreme temperatures. This PPE is not designed to protect against high-speed particles at both ambient and extreme temperatures. We warn that eye protectors against high-speed particles worn over reference ophthalmic glasses can transmit impacts, thus generating a danger for the wearer. Furthermore, it is noted that, to ensure protection against high-speed particles at extreme temperatures, the chosen eye protector should be marked with the letter T immediately after the impact letter, i.e. FT, BT or AT. If the impact letter is not followed by the letter T, the eye protector should only be used against high speed particles at room temperature. It is recommended to replace the screens in case of scratches or damage. No special packaging is required for the transport of PPE, a plastic bag or a cardboard box can be used. Use for the device only original Goldplast screens cod. DPGR1CE-21. Goldplast cannot be held responsible for the use of non-original spare parts.

AREAS OF USE: protection against droplets and splashes of liquids (3)

CLEANING, SANITIZING AND STORAGE

After each use, check the integrity of the PPE and, if necessary, clean it with soap. Sanitize the device with alcohol or hydroalcoholic solution (e.g. 70% alcohol spray). Do not use abrasive detergents or disinfectants. Store unused PPE in a dry environment and at a temperature between 10 °C and 40 °C. We recommend replacing the PPE after 3 years from the date of manufacture - specified on the label of the package.

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