Face Shield SP3224

User Manual

The face shield is designed to protect the wearers eyes, nose and mouth from splashes, sprays, and spatter of liquids. Please read this manual carefully before usage!

Warning

The user manual shall be made available to all users by posting it where the face shields are distributed or by other means.
The face shield should be checked before each use. Scratched or damaged face shields should be disposed of. It is not possible to repair the face shield.
The headband and foam, when in contact with skin, can cause allergic reactions to susceptible users.

Applicable laws and approvals

The product is designed and tested to fulfil the requirements set forth in PPE EU 2016/425 as a category II PPE and with applicable parts of standard EN166:2001.
Testing and type certification have been made by Intertek, notified Body no. 0362, certificate number LECFI00381157.

Instructions for use

Remove the protection film before first use. Hold the face shield against the face and pull the head band over your head.

Cleaning

The face shield can be cleaned using a damp cloth with a disinfectant, ethanol or isopropanol is recommended.

NB! Do not dip or soak the foam with alcohol or disinfectants as it will damage the foam.

Storage

Storage should be made in a clean, dry place to avoid dust, scratching, and unnecessary wear. Do not store in direct sunlight or in temperatures above 30 degrees or below 5 degrees Celsius. Expected shelf time in un-opened packaging is more than 2 years.

Recycling

The packing should be recycled as the inner-bag as soft plastic (PE).
The face shield should be disposed as combustible waste.

Manufacturer

This product is manufactured by Taizhou Shubeikang medical Technology Co., Ltd, Yunhu Village, Hengjie Town, Luqiao District, Taizhou City, Zhejiang, China.

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Fabrikant

Dette produkt er produceret af Taizhou Shubeikang medical Technology Co., Ltd, Yunhu Village, Hengjie Town, Luqiao District, Taizhou City, Zhejiang, China.