Disposable Mouth Guard User Manual

The mouth guard is designed to cover nose and mouth.
Please read this manual carefully before usage!

Warning


The user manual shall be made available to all users by posting it where the mouth guards are distributed or by other means.
The mouth guard should be checked before each use. Scratched or damaged mouth guards should be disposed of. It is not possible to repair the mouth guard.
The ear bands and plastic, when in contact with skin, can cause allergic reactions to susceptible users.

Applicable laws and approvals

Tested and approved according to the Regulation (EC) No 1935/2004 of the European Parliament and of the Council of 27 October 2004, Commission Regulation (EU) No 10/2011 of 14 January 2011 on Overall migration, Sensorial examination odour and taste test, Specific migration of Ethylene glycol & Diethylene glycol, Specific migration of Antimony, Specific migration of Acetaldehyde, Specific migration of Terephthalic Acid, Specific migration of Heavy Metal, and affects all producing, importing, using, or placing products on the European market.
Disposable mouth guards we provide are comply with (EU) No 10/2011.

Instructions for use

Remove the protection film before first use. Adjust the ear bands. Hold the mouth guard against the mouth and nose and pull the ear bands around your ears.

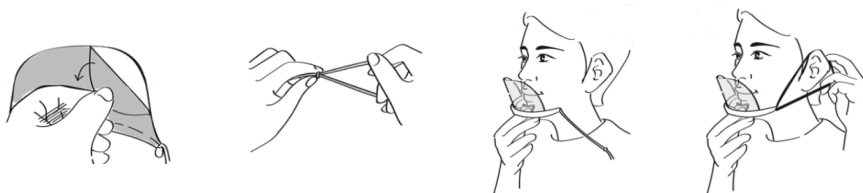Cleaning

The mouth guard can be cleaned using a damp cloth with a disinfectant, ethanol or isopropanol is recommended.
NB! Do not dip or soak the mouth guard with alcohol or disinfectants as it will damage the product.

Storage

Storage should be made in a clean, dry place to avoid dust, scratching, and unnecessary wear.
Do not store in direct sunlight or in temperatures above 30 degrees or below 5 degrees Celsius.
Expected shelf time in un-opened packaging is more than 2 years.

Recycling

The packing should be recycled as the inner-bag as soft plastic (PE).
The mouth guard should be disposed as combustible waste.

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