# ENFit®

LOW DOSE ENTERAL  
SYRINGE WITH ENFit  
CONNECTOR FOR  
ENTERAL USE.

## INSTRUCTIONS FOR USE

### DEVICE DESCRIPTION

Enteral syringe with ENFit connector.

### INTENDED USE

Medicina ENFit Enteral syringes can be used for delivery of enteral feeding, aspiration, flushing and to deliver enteral medicines to all patients groups requiring assistance for feeding. There are two types of ENFit syringe, one designed for hospital use and the second is a reusable syringe for home care patients, both are designed to reduce the risk of enteral tube misconnection.

### INDICATIONS FOR USE

ENFit syringes connect to gastrointestinal delivery system systems such as gastric / nasogastric devices to deliver nutritional formulations or medicine to patients physically unable to eat or swallow for various physiological conditions. It can also be used for flushing the gastric content of patients.

### INTENDED USER

Medical personnel and/or appropriately trained individuals for home use.

### CONTRAINdications

The syringe is not intended for parenteral use.

### WARNINGS/ PRECAUTIONS

- Do not re-use or reprocess as this may lead to leakage or infection.
- The syringe is intended for use on a single patient only.
- Do not use if the packaging is open, syringe tip is visibly blocked, or the syringe is damaged.
- Should fluid get into the tip of the syringe or the moat of the Low dose syringes then it should be removed by tapping the syringe. If used with a pump system improper selection of syringe or syringe profile in a pump menu may lead to inaccurate delivery to the patient.

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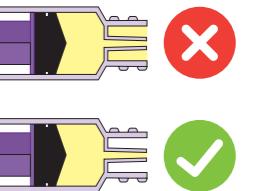
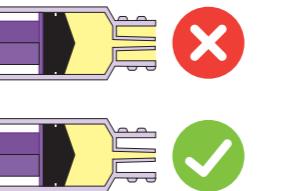
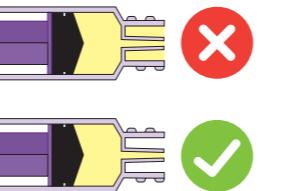
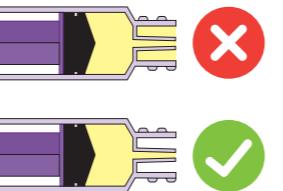
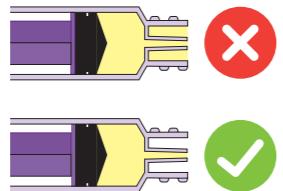
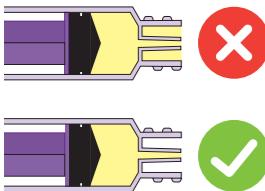
#### DIRECTIONS

- Always follow the protocol for using Enteral syringes as advised by the medical healthcare professional.
- Syringe must be connected to a male ENFit compatible connector during filling, delivery AND any other plunger operation to reduce the risk of dosing volume inaccuracies. Medicina does not recommend the use of the "cupfill" technique for filling syringes.
- Fully depress the plunger before filling. Draw liquid into syringe. For accurate measurement use the leading edge of the black gasket. Do not fill beyond the maximum graduation.
- To expel air bubbles, hold syringe with tip pointing up. Tap side of syringe to allow bubbles to rise toward tip. Draw back slightly on plunger, then push plunger upwards to eject air into the barrel and tip. Repeat until bubbles are eliminated.
- Use immediately or cap off and label syringe until ready to use.

#### DISPOSAL

Dispose of syringe as required by Hospital Protocols. If used in home, dispose in household rubbish, or contact the local state healthcare agency

#### STORAGE CONDITIONS

#### LABELLING SYMBOLS

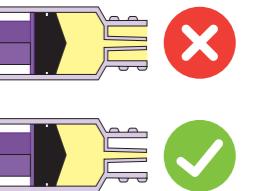
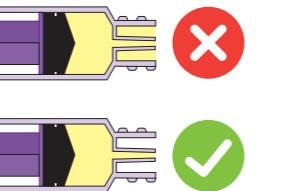
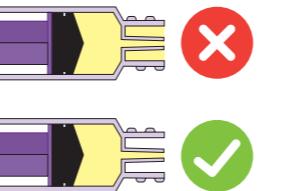
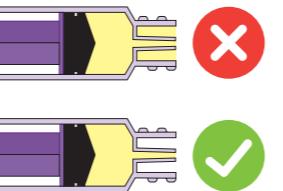
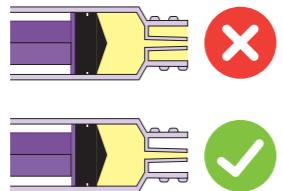
<b>STERILE</b>	<b>EO</b>		STERILIZED using Ethylene Oxide
			Single Use      Do not re-sterilize
			Caution      Consult instructions for use
			Caution: this device is for sale by or on order of a physician Not use if packaging is damaged Not made with phthalates or BPA Does not contain Natural Rubber
			HMC Premedical S.p.A. Via Bosco 1/3, 41037 Mirandola (MO) – Italy

**MEDICINA LTD.**  
Unit 2-4, Rivington View  
Business Park, Station Road,  
Bolton BL6 5BN, UK.

**CE**  
1 6 3 9

ENFit is a registered trademark of GEDSA used with their permission.  
ENFITPELDIFU0071639

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