ENFit® nasogastric feeding tube Instructions For Use

DEVICE DESCRIPTION:
The ENFit Nasogastric Feeding Tube equipped with Male ENFit Connector in line with EN ISO 80369-3 European Standard.

These tubes are designed for nasogastric or nasointestinal, continuous or intermittent enteral feeding. Tubes are made of polyurethane or silicone, they are biocompatible and radiopaque, or have a radiopaque line to be easily located by X-rays. The tubes are graduated to ease insertion and control any possible displacement.

The long-term tubes NGP range have guide wires and the short-term tubes CG & RT ranges do not.

INDICATIONS AND INTENDED USE:
The short-term tubes (CG) are mainly used in neonates and children but can be used in adults who need short-term enteral support.

Short-term wide bore tubes (RT) are used to aspirate and feed adults and children. Typically they are used in intensive care, high dependency units, theatres, crash trolleys and surgical wards.

Short-term tubes should not be used for longer than 30 days.

Long-term tubes (NG) are polyurethane tubes with guidewires. They are used for long term support to feed adults and children. Long term tubes should not be used for longer than 60 days

When necessary the device should be replaced.

CONTRAINDICATIONS:
Do not use Nasogastric Tubes in the case of:

- (1) Gastrointestinal failure.
- (2) Intestinal obstruction.
- (3) Acute abdomen.
- (4) Active gastrointestinal bleeding.

INTENDED USER:
The device must be used exclusively by healthcare professionals, or suitably trained personnel.

STORAGE CONDITIONS

Keep Dry

Keep away from Sunlight

DISPOSAL:
For the disposal of the Nasogastric Tubes follow the local Hospital/Healthcare facility protocols or as directed by the Healthcare professional.

ENTERAL USE ONLY CAREFULLY READ THE INSTRUCTIONS BEFORE USE.

CAUTION: FEDERAL LAW (USA) restricts this device to sale by or on the order of a physician. Follow Physicians instructions using this device.

Medicina Ltd.
Units 2-4, Rivington View Business Park,
Station Road, Bolton BL6 5BN UK
Tel: 01204 695050

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