ENFit®

**REUSABLE LOW DOSE
ENTERAL SYRINGE WITH
ENFit CONNECTOR FOR
ENTERAL USE.**

INSTRUCTIONS FOR USE

DEVICE DESCRIPTION

Enteral syringe with ENFit connector.

INTENDED USE

Medicina ENFit Enteral syringes can be used for delivery of enteral feeding, aspiration, flushing and to deliver enteral medicines to all patients groups requiring assistance for feeding. There are two types of ENFit syringe, one designed for hospital use and the second is a reusable syringe for home care patients, both are designed to reduce the risk of enteral tube misconnection.

INDICATIONS FOR USE

ENFit syringes connect to gastrointestinal delivery system systems such as gastric / nasogastric devices to deliver nutritional formulations or medicine to patients physically unable to eat or swallow for various physiological conditions. It can also be used for flushing the gastric content of patients.

INTENDED USER

Medical personnel and/or appropriately trained individuals for home use.

CONTRAINDICATIONS

The syringe is not intended for parenteral use.

WARNINGS/ PRECAUTIONS

- Stop the use of this re-usable syringe past 7 days from its first use or 40 washes (whichever is sooner), as this may lead to leakage or infection after 7 days / or 40 washes period is expired.
- The syringe is intended for use on a single patient only.
- Do not use if the packaging is open, syringe tip is visibly blocked, or the syringe is damaged.
- Discard after a maximum of 7 days/40 washes.
- Should fluid get into the tip of the syringe or the moat of the Low dose syringes then it should be removed by tapping the syringe. If used with a pump system Improper selection of syringe or syringe profile in a pump menu may lead to inaccurate delivery to the patient.

DIRECTIONS

- Always follow the protocol for using Enteral syringes as advised by the medical healthcare professional.
- Syringe must be connected to a male ENFit compatible connector during filling, delivery AND any other plunger operation to reduce the risk of dosing volume inaccuracies. Medicina does not recommend the use of the "cupfill" technique for filling syringes.
- Fully depress the plunger before filling. Draw liquid into syringe. For accurate measurement use the leading edge of the black gasket. Do not fill beyond the maximum graduation.

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- To expel air bubbles, hold syringe with tip pointing up. Tap side of syringe to allow bubbles to rise toward tip. Draw back slightly on plunger, then push plunger upwards to eject air into the barrel and tip. Repeat until bubbles are eliminated.
- Use immediately or cap off and label syringe until ready to use.

CLEANING INSTRUCTIONS

Clean the syringe immediately after use – failure to do so may result in the syringe becoming contaminated with dry feed or medication which may cause problems if mixed with fresh fluids. Pull the plunger to the end of the barrel and gently manipulate the plunger from side to side while pulling gently – the plunger will come out of the barrel. Clean the barrel and the plunger in warm water – do not use any other cleaning fluids. Pay particular attention to the plunger seal and the nozzle of the barrel. Rinse the barrel and the plunger in cold tap water. Place on clean fresh kitchen roll or paper towel to dry. Avoid touching the seal of the plunger or the inside surface of the barrel. To reassemble – inspect the barrel and plunger ensuring that there are no traces of feed or medicines – check that the inside of the barrel is free from scratches or abrasions and the surface of the seal is not damaged or distorted – it must lie flat and secure in the groove on the plunger. Then gently manipulate the plunger back into the barrel and push the plunger the length of the barrel until its face almost touches the other end of the barrel. Store the re-assembled syringe in a dry container ready for the next use. If the syringe cannot be cleaned effectively or is damaged, discard it and use a fresh one. The cleaning and rinse water can drain away as normal – there is no need for any special precautions.

The syringe must only be used for a maximum of 7 days or 40 washes before it is discarded and a fresh one used. The syringe may not operate effectively if used after this seven day period has expired. Do not attempt to use the syringe if the markings are unclear or the syringe is stiff or difficult to operate. Clean the syringe prior to disposal and dispose of the syringe as per the Disposal instructions.

DISPOSAL

Dispose of syringe as required by Hospital Protocols. If used in home, dispose in household rubbish, or contact the local state healthcare agency.

STORAGE CONDITIONS

LABELLING SYMBOLS

HMC Premedical S.p.A
Via Bosco 1/3,
41037 Mirandola (MO) – Italy

MEDICINA LTD.
Unit 2-4, Rivington View
Business Park, Station Road,
Bolton BL6 5BN, UK.

ENFit is a registered trademark of GEDSA used with their permission.
ENFITHELDIFU0071639

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