## MMA20 Technical File.

### MMA-900-0094 produktoplysninger.

<b>Part Number/ Varenummer:</b>	MMA-900-0094
<b>Part Description/ Varenavn:</b>	Enema Tip, for single colon investigation and irrigation AquaColon Rektalkateter med ballon
<b>Date/Dato/Version</b>	09/08/2022
<b>Reference number/ Varenummer :</b>	MMA-900-0094
<b>Intended use/ Anvendelse:</b>	Part of a kit for a delivery Device for Colon investigation or Irrigation Til brug for undersøgelse af colon/tyktarm eller irrigation af tyktarmen
<b>Colour/ Farve:</b>	Violet Lilla
<b>Size/ Størrelse:</b>	680.0mm
<b>Weight/ Vægt :</b>	25g
<b>Classification/ Medicinsk udstyr:</b>	Medical Device class 1 Medicisk udstyr klasse 1

**Medical product class:** Class 1 non sterile  
An invasive device which is intended  
for transient use, as defined MDR ( EU ) 2017-745

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**Country of Origin:** United Kingdom  
**Oprindelsesland:** England

**Manufacturer:**  
**Producent** Runfold Plastics Ltd  
Passfield Mill Business Park  
Liphook  
Hampshire  
GU30 7RP

**Contact: Telephone**  
**Telefonnummer** +44 (0)1428 753320  
**E mail:** [sales@runfoldplastics.co.uk](mailto:sales@runfoldplastics.co.uk)  
**Web:** [www.runfoldplastics.co.uk](http://www.runfoldplastics.co.uk)

## English (EN) User Instructions

### Description: Colon Tubes

MMA-900-0094

Single contrast colon tip, step adaptor for enema bag connection. Cuff silicone balloon for retention

**Selection of the colon tube is made by an appropriate medical professional, and is subject to patient requirements. Please note the information provided only covers the colon tubes as part of a transient use for the procedure.**

#### Contraindications:

This product should not be used if the patient has any of the following known conditions. The list is not exhaustive, and a healthcare professional should always consider the patient's medical history. It should always be administered by a medical professional.

- Known anal or colorectal stenosis
- Colorectal cancer pre-surgical removal
- Acute inflammatory bowel disease or acute diverticulitis
- Within recent anal or colorectal surgery
- Recent endoscopic polypectomy
- Ischemic colitis

#### Warnings:

- When the product with a cuff balloon is fitted, care must be taken not to over inflate the balloon cuff. Over inflation could cause the cuff to rupture with possible injury to the patient. Maximum 85ml of air is allowed into the balloon.
- Forceful or deep insertion may cause tearing or perforation of the bowel.
- The colon tip should not be removed unnecessarily once inserted.
- The colon tubes are intended for single use.
- Reuse of the product is a potential risk to the patient, including a risk of infection. Contamination of the product can lead to injury, illness or in extreme cases death of the patient. Reusing the product may also compromise the integrity of the medical device.
- Never block the balloon with too much air, as colon necrosis can develop.

#### Precautions and allergic reactions:

- Care to be taken during insertion.
- When fitted, inflation of the silicone balloon retention cuff must only be performed by a medical professional.
- Patients notes must be reviewed for any potential rare allergic reactions.

#### Intructions for use:

- The end of the colon tip should be lubricated with a suitable lubricant, ideally water based.
- Insert the tip of the colon tube into the anus carefully, until the tip or retention cuff is just beyond the anorectal junction.
- When using a colon tube with a cuff silicone balloon, when the balloon is inflated the base of the cuff should be at the level of the anal sphincter. Care should be taken not to over inflate – see **Warnings** and **Precautions**.
- Certain medical practices and some countries' regulations recommend water to be use when inflating the balloon. This is allowed but only to the maxumium of 60ml of water.
- After use, discard the tube using the correct approved procedure. (The product is single use only.)

### **Storage condition requirements;**

For best results store the product:

- Sealed and double bagged.
- At room temperature, about 20 °C (68 °F) - avoid high temperatures
- Keep product out of direct sunlight

### **Transport condition requirements;**

- Product is to remain stored in its original sealed packaging.
- Avoid exposure to direct sunlight and high temperatures.

### **Shelf Life;**

For best results the product should be used within one year from the date of manufacture.  
If stored correctly the product will function appropriately after 4 years from the date of manufacture.

### **Recommend user Guidance;**

- Single use application – do not reuse 
- Maximum balloon capacity of 85ml (when a balloon is fitted)
- Must be operated by a Medical professional
- Patient condition to be fully assessed by a medical professional to avoid risk of harm including potential bowel injury.

### **RoHS 3 DIRECTIVE (EU 2015/863)**

- This product does not use any substances currently listed in RoHS3 Directive 2015/863/EU. According to our supplier information and to the best of our knowledge, the raw materials used to manufacture the product meet the acceptance tolerances currently listed in RoHS3 Directive 2015/863/EU.

### **REACH (Registration, Evaluation, Authorization and Restriction of Chemicals)**

- Refer to individual component details for relating SVHC specifics.

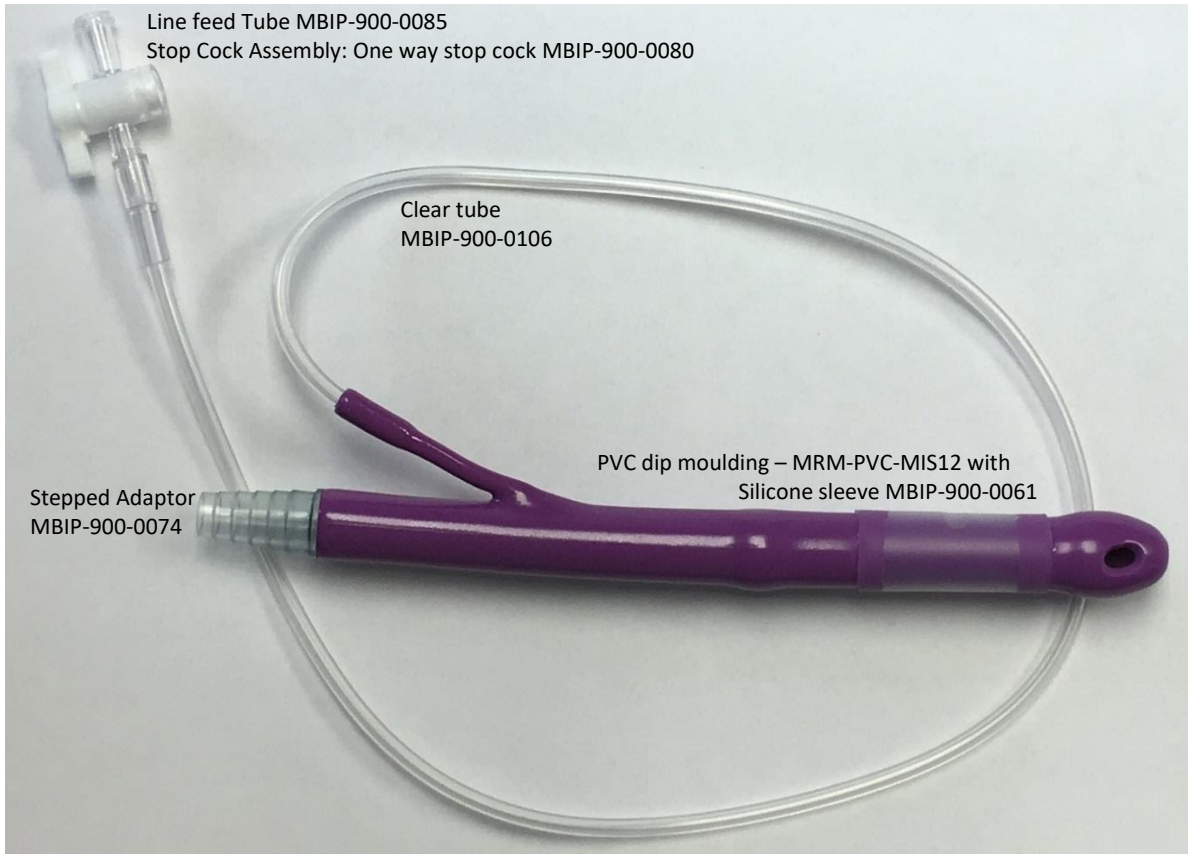
### **Latex or Materials Containing Latex**

- This product does not contain Latex. According to supplier information and to the best of our knowledge, the raw materials used to manufacture the product do not contain Latex.

The finished device is fully assembled using the following components:

Part Description	Part Number	Quantity
Probe	MRM-PVC-MIS12	1
Silicone Sleeve	MBIP-900-0061	1
Stepped Adaptor	MBIP-900-0074	1
Clear Tube	MBIP-900-0106	1
Line feed tube	MBIP-900-0085	1
One way stop cock	MBIP-900-0080	1

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Kirstine Hardam A/S