
Microlife Cuff

IB 3G-cuff all-sizes V32 1014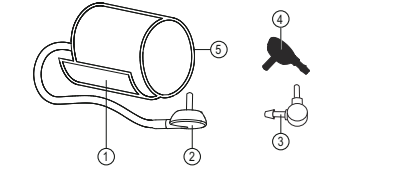

Description
<div> <div>① Cuff</div> <div>② Connector A</div> <div>③ Connector B (only in S, M, and M-L)</div> <div>④ Connector C (only in L and L-XL)</div> <div>⑤ Removable bladder (not in the 22-42 cm conical rigid cuff)</div> </div>

Choosing the connector

Connector A ⓘ use for automatic blood pressure monitors (a motor pumps the cuff) which have a yellow colored air connection socket.
Connector B ⓘ for **S, M, M-L cuffs** or **connector C** ⓘ for **L, L-XL cuffs**: use for automatic devices which DO NOT have a yellow colored air connection socket and for semi-automatic blood pressure monitors (with pump ball).

No connector: Semi-automatic blood pressure monitors which have a tube shaft directly on the device. Simply plug the tube to that shaft.

Cleaning the cuff

Carefully remove spots on the cuff with a damp cloth and soapsuds.
Soft cuff sleeve: Always remove the bladder ☞ from the sleeve before washing. Afterwards precisely reinstall the bladder.

⚠ WARNING: Under no circumstances may you wash the inner bladder!

⚠ WARNING: Do not wash the cuff in a washing machine or dish-washer!

Guarantee

The cuff has a functional guarantee (bladder lightness) for 2 years.

- Not included are sleeve and cuff enclosure due to wear.
- The guarantee is only valid on presentation of the purchase receipt.
- The guarantee does not cover damage caused by improper handling, accidents or non-compliance with the operating instructions.

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